

## How to import the library into your Exocad system?

1. Close your Exocad software
2. Unzip the folder “zeramex\_p6\_exocad\_v2.zip”
3. Move the unzipped folder to the following installation directory of the Exocad software: \DentalCADApp\library\implant
4. Launch your Exocad software
5. You have successfully installed the ZERAMEX® P6 ZERABASE library.

## How to import the library into your 3Shape system?

1. Open the 3Shape Dental System Control Panel
2. Tools ► Import/Export
3. Select “zeramex\_p6\_v3”. Save and close the “3Shape Dental System Control Panel”.
4. You have successfully installed the ZERAMEX® P6 ZERABASE library.

Please **contact the support service of your CAD/CAM provider with any questions** on software data import.

Order the parts necessary for treatment directly with ZERAMEX®.

### Order-Hotline

T 00800 – 93 55 66 37

F 00800 – 93 55 63 77

[order@zeramex.com](mailto:order@zeramex.com)